

ELECTRICAL SCOOTER



WT-T4Q

OWNER'S MANUAL

Issued on May 6, 2019
Version: 1.0

TABLE OF CONTENTS

1.	SAFETY GUIDELINES	2
2.	SAFETY NOTICE	3
3.	PARTS INTRODUCTION	8
4.	FOLDONG OUT/UP THE SCOOTER	9
5.	OPERATION	10
6.	BATTERY CHARGING AND CARE	12
7.	INSPECTION AND MAINTENANCE	15
8.	TROUBLE SHOOTING AND SPECIFICATION	17
9.	WARRANTY	19



1. Always use a seat belt and keep your feet on the scooter at all time.
2. Never operate the scooter while you are under the influence of alcohol.
3. Never use mobile phones or radio transmitters such as walkie-talkies.
4. Make sure that there are no obstacles behind you while reserving your scooter.
5. Do not ride your scooter in traffic.
6. Do not make a sharp turn or a sudden stop while riding your scooter.
7. Do not attempt to climb kerbs greater than limitation shown in the specification.
8. Do not remove your hands and legs from the scooter when driving.
9. Do not ride your scooter during snow in order to avoid accidents.
10. Do not allow unsupervised children to play near this equipment while the batteries are.

1. SAFETY GUIDELINES

Please carefully read this owner's manual before using the scooter. Improper use of the scooter could result in harm, injury or traffic accidents. Therefore for maximum pleasure while using the scooter please read this owner's manual.

- ◆ This owner's manual includes operation instructions for every aspect of the scooter, assembly instructions, as well as instructions for how to deal with possible accidents.
- ◆ The symbols used in this manual are explained below:
Read especially the notes marked with these symbols:



WARNING! Indicates a potentially hazardous condition/situation. Failure to follow designated procedures can cause either personal injury, component damage, or malfunction. On the product, this icon is represented as a black symbol on a yellow triangle with a black border.



MANDATORY! These actions should be performed as specified. Failure to perform mandatory actions can cause personal injury and/or equipment damage. On the product, this icon is represented as a white symbol on a blue dot with a white border.



PROHIBITED! These actions are prohibited. These actions should not be performed at any time or in any circumstances. Performing a prohibited action can cause personal injury and/or equipment damage. On the product, this icon is represented as a black symbol with a red circle and red slash.

- ◆ This manual includes a copy of repair and maintenance record chart and warranty information. Please keep it in a safe place or in the scooter.
- ◆ If someone else uses the scooter, please make sure that you provide them with the instruction manual for his or her consideration.
- ◆ As designs change some illustrations and pictures in the manual may not correspond to the scooter that you purchased. We reserve the right to make design modifications.

Our scooters have been designed and manufactured to provide a comfortable and secure yet affordable solution for some mobility requirements.



Suggestion

To maximize your batteries efficiency and service life, please fully recharge your new battery before its first time use.

2.SAFETY NOTICE

GENERAL



MANDATORY! Do not operate your new Scooter for the first time without completely reading and understanding this owner's manual and the Consumer Safety Guide.

Your Scooter is a state-of-the-art life-enhancement device designed to increase mobility. We provide an extensive variety of products to best fit the individual needs of the Scooter user. Please be aware that the final selection and purchasing decision regarding the type of Scooter to be used is the responsibility of the Scooter user who is capable of making such a decision and his/her healthcare professional (i.e., medical doctor, physical therapist, etc.).

The contents of this manual are based on the expectation that a mobility device expert has properly fitted the Scooter to the user and has assisted the prescribing healthcare professional and/or the authorized Provider in the instruction process for the use of the product.

There are certain situations, including some medical conditions, where the Scooter user will need to practice operating the Scooter in the presence of a trained attendant. A trained attendant can be defined as a family member or care professional specially trained in assisting a Scooter user in various daily living activities.

As you begin using your Scooter during daily activities, you will probably encounter situations in which you will need some practice. Simply take your time and you will soon be in full and confident control as you maneuver through doorways, on and off elevators, up and down ramps, and over moderate terrain.

Additional general information can be found on the supplemental information sheets and booklets included in your Owner's Package. Please fully read and review the information, and keep it readily available for future reference.

Below are some precautions, tips, and other safety considerations that will help you become accustomed to operating the Scooter safely.

■ PRE-RIDE SAFETY CHECK

Get to know the feel of your Scooter and its capabilities. We recommend that you perform a safety check before each use to make sure your Scooter operates smoothly and safely.

■ Perform the following inspections prior to using your Scooter:

- ◆ Check the condition of the tires. Make sure they are not damaged or excessively worn.
- ◆ Check all electrical connections. Make sure they are tight and not corroded.
- ◆ Check the battery condition meter to ensure the batteries are fully charged.

If you discover a problem, contact your authorized Provider for assistance. Please refer to the Contact Information insert in your Owner's Package.

2.SAFETY NOTICE

WHILE DRIVING

- **Please carry out daily inspections.** Refer to the section entitled “DAILY CHECKING”

- **Do not move your body out of the scooter while moving**
 - ◆ Such action may cause you to lose balance and risk injury from falling.
 - ◆ Pay attention that your clothes do not tangle in the wheels.

- **Do not use your scooter under the circumstances below.**
 - ◆ Do not on the roads with heavy traffic or roads that are muddy, gravelly, bumpy, narrow, snowed over, icy, or canal towpaths not guarded by any fence or hedge. Keep away from places where you might get the wheels stuck.
 - ◆ Do not drive at night or when it is raining, snowing, misty, or windy.
 - ◆ Do not drive your scooter in an “S” pattern or make a sharp turn.
 - ◆ Do not take the scooter onto escalators.
 - ◆ UNDER NO CIRCUMSTANCES SHOULD THE SCOOTER BE USED AS A SEAT IN A MOTOR SCOOTER (E.G. CARS, BUSES, TRAINS, ETC).

- **About Mobile Phones and other electrical equipment**
 - ◆ Do not use a mobile phone or other wireless communication devices while driving.
 - ◆ Always switch off the scooter and remove the ignition key before using a mobile phone.
 - ◆ Do not charge the mobile phone or any other electrical devices from your scooter’s battery.

- **Automatic Power Shut Down**

In order to avoid accidental battery run down, Turn off the power without using it.
Should this occur, simply on and it will be ready to use once again.

- **Ramps, inclines and drops**
 - ◆ Do not drive onto steep ramps greater than the specified gradient. Refer to the section entitled “CLIMBING ANGLE” in “7. SPECIFICATION“
 - ◆ Always use a low speed setting when ascending or descending a gradient.
 - ◆ Do not drive on roads with large drops or potholes.
 - ◆ Please slow down when driving on roads with inclines.
 - ◆ Do not make sudden turns when driving on gravel roads or ramps.
 - ◆ Always lean forward when climbing a steep gradient.

2.SAFETY NOTICE

■ Starting and Driving

1. Make sure the seat is installed properly.
2. Make sure the tiller has been secured properly.
3. Check battery indicator to see whether there is enough power for your journey. If you have any doubt about the remaining power, please recharge the batteries before departure.
4. Set the speed dial to a position you feel safe and comfortable with.
5. Check the forward speed works correctly.
6. Make sure the battery works correctly.

Make sure it is safe around you before you drive on the street.

WARNING!



- To protect your safety, the power will consumption and battery system will while you are driving a steep gradient. This will limit the speed to a safe level. Turn the power on again to re-start your scooter.

Maximum User Weight Limit

- Refer to section entitled “MAX. USER WEIGHT” in “ SPECIFICATION “. Overloading past the weight limit may lead to damage of your scooter or cause it to malfunction and will endanger your safety. The warranty does not cover this type of damage.

Attention



1. Do not turn the power switch to OFF while driving as this will lead to an emergency stop and possible risk of accident and injury.
2. Do not set to the highest speeds while driving indoors.
3. Do not adjust the direction button while driving, a sudden change in speed may cause danger to you and others, and may cause damage to your scooter.
4. Do not place magnetic devices near the area of the operation handle as this could affect the safe operation of your scooter.
5. Do be careful while driving in heavy traffic or crowded areas.
6. While reversing the scooter, beware of people or objects behind you.

2.SAFETY NOTICE

■ Stopping

1. Release the speed control lever completely. The scooter will naturally brake and stop.
2. Turn the power switch to (OFF). Then pull out the key.

■ Driving on the pavement

- ◆ Ensure the 6 km/h limit switch is set to the slowest setting when using the scooter on the pavement.
- ◆ This will limit the scooter to a maximum speed of 6.4 km/h. It is an offence to travel at more than 6.4 km/h on the pavement.
- ◆ The limit switch can be set to the fastest setting for use on the road or on private ground.

■ Motor Vehicle Transport

Wu's recommend that you do not remain seated in your scooter while traveling in a motor vehicle. The scooter should be stowed in the trunk of a car or in the back of a truck or van with the batteries removed and properly secured. In addition, all fold scooter parts, including, seat, and support rod should be removed and/or properly secured during motor vehicle transport.

WARNING! Do not sit on your scooter while it is in a moving vehicle.



WARNING! Always be sure your scooter and its batteries are properly secured when it is being transported. Batteries should be secured in an upright position and protective bandage should be installed on the battery terminals. Batteries should not be transported with any flammable or combustible items.

■ Lift/Elevation Products

If you will be transporting your scooter during travel, you may find it necessary to use a vehicle-mounted lift system or elevation product to aid in transportation. Wu's recommends that you closely review the warnings, instructions, specifications, and safety information set forth by the manufacturer of the lift/elevation product before using that product.



WARNING! Always turn off the power to the unit when you are being transferred via an elevation product. Failure to do so may result in accidental throttle control lever contact, resulting in unintended movement and personal injury or product damage.

■ Stairs and Escalators

Scooters are not designed to travel up or down stairs or escalators. Always use an elevator.



WARNING! DO not use your scooter to negotiate steps or escalators.

2.SAFETY NOTICE

EMI

This portion of the content will provide the user with basic information that describes the problems with EMI, known sources of EMI, protective measures either to lessen the possibility or exposure or to minimize the degree of exposure, and suggested action should unexpected or erratic movement occur.



Caution:

It is very important that you read this information regarding the possible effects of electromagnetic interference on your electric scooter.

■ ELECTROMAGNETIC INTERFERENCE (EMI) FROM RADIO WAVE SOURCES

Powered scooter may be susceptible to electromagnetic interference (EMI), which is interfering electromagnetic energy (EM) emitted from sources such as radio stations, TV stations, amateur radio (HAM) transmitters, two-way radios, and cellular phones. The interference (from radio wave sources) can cause the powered scooter to release its brakes, move by itself, or move in unintended directions. It can also permanently damage the powered scooter's control system. The intensity of the interfering EM energy can be measured in volts per meter (V/m). Each powered scooter can resist EMI up to a certain intensity. This is called its "immunity level". The higher the immunity level, the greater the protection. At this time, current technology is capable of achieving at least a 20 V/m immunity level, which would provide useful protection from the more common sources of radiated EMI. This powered scooter model as shipped, with no further modification, has an immunity level of 20 V/m without any accessories.

There are a number of sources of relatively intense electromagnetic fields in the everyday environment. Some of these sources are obvious and easy to avoid. Others are not apparent and exposure is unavoidable. However, we believe that by following the warning listed below, your risk to EMI will be minimized.

The sources of radiated EMI can be broadly classified into three types:

1. Hand-held portable transceivers (transmitter-receivers with the antenna mounted directly on the transmitting unit. Examples include: citizens band (CB) radios, " walkie talkie", security, fire, and police transceivers, cellular telephones and other personal communication devices.
2. Medium-range mobile transceivers, such as those used in police cars, fire trucks, ambulances and taxis. These usually have the antenna mounted on the outside of the Scooter,
3. Long-range transmitters and transceivers, such as commercial broadcast transmitter (radio and TV broadcast antenna towers) and amateur (HAM) radios;

3.PARTS INTRODUCTION

PARTS DESCRIPTION

- | | | | |
|-------------------------|-------------------|--------------------|------------------------|
| 1. SEAT | 2. Control Panel | 3. Slide Lever | 4. Tiller Swivel Knob |
| 5. Front Footrest | 6. Rear Reflector | 7. speed/slow down | 8. Switch speed button |
| 9. Charger Socket Cover | | | |



4.FOLDING UP/OUT THE SCOOTER


1.Pull the yellow knob back to the rear support bar.



6.Push the seat back (Folding up the scooter p.3)



2.Push the support rod forward to make the bottom plate smooth, press the pedal.

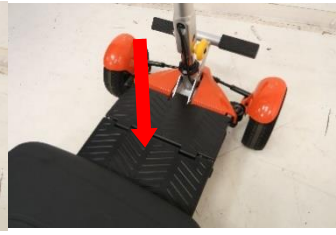

7. Push up the pedal, position the pedal forward. (Folding up the scooter p.2)



3.Pull the seat yellow push back, lift up placement



8. Pull the yellow knob back to the rear support bar. (Folding up the scooter p.1)



5.Complete.



10.Complete.



5. OPERATION

5.1 CONTROL PANEL

1. Battery Level Indicator
2. Speed Lever
3. Forward / Reverse Selector



PROHIBITED! Do not expose the tiller console to moisture. In the event that the tiller console does become exposed to moisture, do not attempt to operate your Scooter until the tiller console has dried thoroughly.

5.2 HOW TO OPERATE YOUR SCOOTER

■ Power switch

Insert or pull out the key to power ON or OFF

(ON) : Power is turned on

(OFF) : Power is turned off



WARNING!

If the key is moved to the “off” position while your Scooter is in motion, the electronic brakes will engage and your Scooter will come to an abrupt stop!

5. OPERATION

■ Forward, Reverse, and Braking

- ◆ The button control upwards with your left fingers and pull the scooter will move forward.
- ◆ The button control downwards with your left fingers and pull the scooter will move backward.
- ◆ Release the speed control lever freely while either going forward or reverse, and the electromagnetic brake in the motor will be activated, and the scooter will stop.

■ Battery Indicator

- ◆ When the power switch is turned on the battery indicator will light up with show color squares the remaining battery power. The “green” means full power, the “red” means off power.
- ◆ The remaining power indicated by the battery indicator will vary by the actual driving time incurred and how you drive. Repeated starting, stopping, climbing will consume the power more quickly.



The remaining power suggested by the battery indicator will vary by the actual driving time incurred and how you drive. Repeated starting, stopping, climbing will consume the power more quickly.

If the power down timer feature takes effect, perform the following steps to resume normal operation.


1. Remove the key from the key switch.
2. Reinsert the key and power up your Scooter.

6. BATTERY CHARGING AND CARE

6.1 CHARGING THE BATTERY

Follow the procedure below step by step:

1. Turn the power switch to (OFF)
2. Connect the charger's power cord into the power outlet.
3. Open the charging socket cap on the tiller. Then connect the charger's round plug to the charging socket.
4. Turn on the switch on the charger.
5. Both the charger's red and orange LEDs will be lit when beginning charging. The charging duration is about 6 hours. To ensure optimum performance a 10-hour charge is recommended. But we do not recommend a charging more than 24 consecutive hours.
6. Both the charger's LEDs will be lit during the charging process. The orange LED will turn green when charging is complete.
7. Turn off the charger; disconnect the power cord and the round plug from charger socket on the scooter.

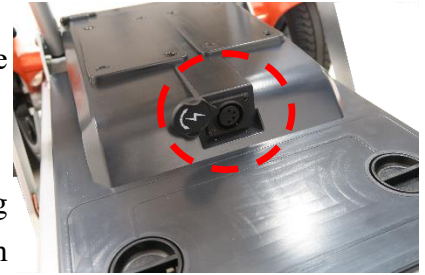

6.2 BATTERY

- ◆ Do not expose the battery to temperatures below 10° C or above 25° C when charging or storing the scooter. Under the above temperature range can cause the battery either to freeze or over heat. This will damage the batteries and shorten their life.
- ◆ These batteries are maintenance free and there is no need to refill with water.
- ◆ You are required to recharge the batteries on a regular basis. Even if the scooter is stood idle, you should charge the batteries at least once a week.
- ◆ The batteries carry a six-month manufacturer's warranty. This warranty only covers issues relating to manufacturing faults, and not faults relating to failure to recharge the batteries as instructed above.

Suggestion – How to maximize your batteries efficiency and service life

1. Fully recharge your new battery before its first time use.
2. Be sure to charge the battery fully every time. The battery life will be seriously shortened or decayed if the battery is repeatedly used without being fully charged.
3. Always complete the charging until the orange LED light turns green. NEVER stop charging before it is complete.
4. Keep your batteries fully charged whenever possible.
5. If you do not use your scooter for a long time, it should be charged at least every week to keep the battery in a fully usable condition.
6. The ambient temperature will affect charging time. Charging time will be longer in the winter.
7. After charging, do not leave the charger socket plugged in to the scooter, as this will cause a power drain on the scooter and temporarily reduce its range.

6. BATTERY CHARGING AND CARE



PROHIBITED! Removal of grounding prong can create electrical hazard. If necessary, properly install an approved 3-pronged adapter to an electrical outlet having 2-pronged plug access.



PROHIBITED! Never use an extension cord to plug in your battery charger. Plug the charger directly into a properly wired standard electrical outlet.



PROHIBITED! Do not allow unsupervised children to play near the Scooter while the batteries are charging. We recommend that you do not charge the batteries while the Scooter is occupied.



MANDATORY! Read the battery charging instructions in this manual and in the manual supplied with the battery charger before charging the batteries.



WARNING! Explosive gases may be generated while charging the batteries. Keep the Scooter and battery charger away from sources of ignition such as flames or sparks and provide adequate ventilation when charging the batteries.

WARNING! You must recharge your Scooter's batteries with the supplied off-board charger. Do not use an automotive-type battery charger.

WARNING! Inspect the battery charger, wiring, and connectors for damage before each use. Contact your authorized Provider if damage is found.



WARNING! Do not attempt to open the battery charger case. If the battery charger does not appear to be working correctly, contact your authorized Provider.

WARNING! If the battery charger is equipped with cooling slots, then do not attempt to insert objects through these slots.

WARNING! Be aware that the battery charger case may become hot during charging. Avoid skin contact and do not place on surfaces that may be affected by heat.



WARNING! If your battery charger has not been tested and approved for outdoor use, then do not expose it to adverse or extreme weather conditions. If the battery charger is exposed to adverse or extreme weather conditions, then it must be allowed to adjust to the difference in environmental conditions before use indoors. Refer to the manual supplied with the battery charger for more information.



6. BATTERY CHARGING AND CARE

■ Cleaning the battery

If the batteries are contaminated by water, battery acid, dust or other substances, they will discharge quickly. The batteries supplied with the WU'S scooter are sealed and as such are maintenance free with no risk of battery leakage. Please follow the steps below to clean the battery.

1. Turn the scooter power switch to “OFF”.
2. Remove the seat and dust cover.
3. Use a clean cloth to wipe off the soiled area.
4. Clean the battery with a clean cloth. If the terminal is covered by white powder, please wipe it clean using warm water.




1. Be sure the battery wires are connected to their right battery terminal.
2. If necessary, ask for help from your dealer for advice about maintaining and replacing the battery.

📖 Suggestions

1. Make sure the terminals are installed properly and put the cover back on.
2. Do not use the battery to charge telecom equipment or other items.
3. Battery efficiency will vary with outside conditions; the driving distances will be shorter in the winter. If the scooter is not used for a long time, please charge the battery at least every week.
4. Remove the power connector for more than 1 month. and the charger can not be dropped
5. For air transportation the battery should be disconnected before transporting.
6. Do not disassemble arbitrarily (very dangerous).

■ To disconnect the battery:

1. Turn the knob and open the cover.
2. Once the cover is removed, unplug the red connector shown below.

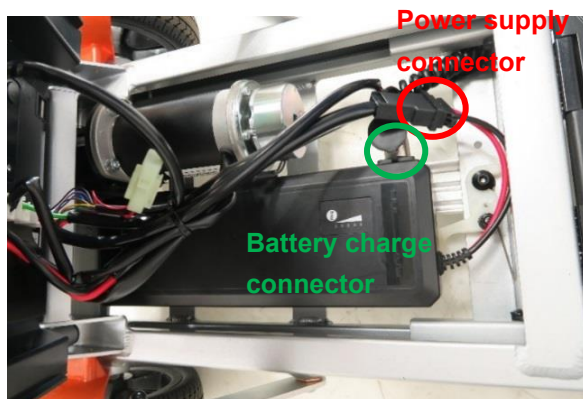

3. Reinstall the battery compartment cover to prevent dust and moisture getting in to the compartment.

7.INSPECTION AND MAINTENANCE

7.1 INSPECTION

- ◆ Clean the scooter with a damp cloth and dust down approximately once a week to preserve the appearance of the scooter.
- ◆ Check for signs of wear and tear on the tires and the upholstery on a regular basis.
- ◆ For optimum performance and to increase the lifespan of your scooter, it is recommended that you have your scooter serviced once a year.

7.2 REGULAR CHECKING RECORD

To make sure your scooter is correctly serviced, take it to your dealer for regular maintenance checks. This should be at intervals of six months after the first inspection after one month. Your dealer may charge a fee for this service.

YEAR	1	2	3	4	YEAR	1	2	3	4
Service Dates					Service Dates				
Controller					Upholstery				
On/off switch					Seat				
Controller Lever					Back				
Braking					Armrests				
Recharge point					Electrics				
Batteries					Connections condition				
Levels					Lights				
Connections					Test run				
Discharge test					Forwards				
Wheels and Tyres					Reverse				
Wear					Emergency stop				
Pressure					Left turn				
Bearings					Right turn				
Wheel nuts					Slope test				
Motors					Over obstacles				
Wiring					List Items repaired				
Noise									
Connections									
Brake									
Brushes									
Chassis									
Condition									
Steering									

7.INSPECTION AND MAINTENANCE

■ Tires

The condition of the tires depends on how you drive and use your scooter.

- ◆ Inspecting tire tread
- ◆ Please check the tread groove regularly. Replace the tires when the tread groove is less than 0.5 mm.



Attention

1. When conducting maintenance of your scooter, please turn the power switch to OFF and remove the charger cords.

📖 Suggestions

- ◆ Do not splash water directly to wash your scooter as this could lead to malfunction of the system electrics.
- ◆ Do not use petrol, solvents or vaporizing solution as these may deform or damage the shrouds.

7.3 STORAGE

Make sure the scooter is stored under the following circumstances:

- Make sure the seat is set in the “Forward” position
- Make sure the Power switch is turned to OFF

📖 Suggestion

Please store the scooter in a location where it is away from the direct sunlight, rain, or dew. When storing for a long time, please charge the battery to full power and then disconnect the battery terminal. For details inquire to your scooter dealer.

7.4 MOVING ABOUT

- ◆ Turn the power switch “OFF” before moving.
- ◆ For your safety, always ask for help if required. You will need 2 or 3 persons when moving or lifting the scooter.

8. TROUBLE SHOOTING AND SPECIFICATION

8.1 TROUBLE SHOOTING

Symptom	Remedy
The scooter will not switch on	<ul style="list-style-type: none">• Try recharging the battery• Check the fuse and circuit breaker in the scooter
The scooter switches on, but the scooter will not move.	<ul style="list-style-type: none">• Ensure there is enough power in the batteries. If not, recharge the batteries.• Ensure the freewheel lever is engaged in the 'DRIVE' position.
The scooter appears slow	<ul style="list-style-type: none">• Check the battery power level and recharge
The seat move when in operation	<ul style="list-style-type: none">• Slowly the seat until it drops in to place and is secure
The handlebar appears loose	<ul style="list-style-type: none">• Tighten the height adjustment handle to secure the handlebar• If problem persists contact your WU'S dealer.

8.2 SPECIFICATION

Scooter Size (L×W×H)	33.5" x 17" × 33.5" / 850 x 430 x 850 mm
Total Weight	17 kg
Battery Weight	1.85 Kg
Weight	0.4 kg
Front Wheel	165 × 50 mm
Rear Wheel	165 × 50 mm
Battery Specifications	24V 11.6 AH
Charger Type	2A Off-Board
Motor Type	DC24V / 180W / 6000 RPM (1:27)
Turning Radius	1400 mm
Max Speed	6.4 km/h
Cruising Range	10 km
Weight Capacity	115 kg
Max Climbing Angle	3°
Controller Type	Dynamic 50A

Remark: The manufacturer reserves the right to modify the specification if necessary. The final specification is subject to the individual scooter you purchase from your dealer.

Note:

Please note that the maximum range is based on an ambient temperature of 25°C, a 17kg driver, a brand New and fully charged battery, and a constant driving speed of 4/5 mph with 70% battery power discharged.

9.WARRANTY

◆ WARRANTY

There is a comprehensive twelve-month warranty from the date on which your new **WT-T4Q** scooter is delivered. The warranty covers the scooter for repairs or replacement during this period. For more detail, please see the Warranty Conditions below.

Warranty Conditions:

1. Any work or replacement part installation must be carried out by an authorized dealer / service agent.
2. To apply the warranty should your scooter require attention please contact the designated service agent listed below.
3. Should any part of the scooter require repair or full or part replacement, as a result of a manufacturing or material defect within warranty period, the work will be carried out free of charge. Warranty period:
 - a)Frame : 3 Years limited warranty
 - b)Electronic parts : 24 months limited warrantyNote: The warranty is not transferable
4. Any repaired or replaced parts will be covered by this warranty for the balance of the warranty period on the scooter.
5. Parts replaced after the original warranty has expired will be covered by a three months warranty.
6. Consumable items supplied will not generally be covered during the normal warranty period unless such items require repair or replacement clearly as a direct result of a manufacturing or material defect.

Such items include (among others): upholstery, tires and batteries.
7. The above warranty conditions apply to brand new scooters purchased at the full retail price. If you are unsure whether your scooter is covered, check with the service agent.
8. Under normal circumstances, no responsibility will be accepted where the scooter has failed as a direct result of:
 - a) The scooter part not having been maintained in accordance with the manufacturer's recommendations.
 - b) Failure to use the manufacturer's specified parts
 - c) The scooter or part having been damaged due to neglect, accident or improper use
 - d) The scooter or part having been altered from the manufacturer's specifications or repairs having been attempted before the service agent is notified

Please note your local service agent's contact details in the box below. In the event of your scooter requiring attention, contact them and give all relevant details so they can act quickly.

The manufacturer reserves the right to alter without notices any weights, measurements or other technical data shown in this manual. All figures, measurements and capacities shown in this manual are approximate and do not constitute specifications.

The authorized Service Agent

Name
Address
Tel
Postcode

9.WARRANTY

9.1 VIN (SCOOTER INDIFICATION NUMBER)

To ensure the correct after-sale service and warranty service support, please write down the scooter identification number that is stuck on the back side of the frame.

Model	WT-T4Q
Chassis NO.	
Motor serial #	

Warranty Application Form

Name			
Tel no.			
Address			
Model	WT-T4Q		
Scooter Data	Scooter No.:	Chassis	
	Motor Serial No.:		
Date of Purchase	Year	Month	Day
Agent Signature			



Wu's Tech Co. Ltd.

**3 Fl. No.112 Sec. 2, Chung-Shan N. Rd.,
Taipei, Taiwan, R.O.C.
Tel: +886 2 2541 3253
Fax: +886 2 2536 7575**

Release Date: 2019/05/6