

IMPORTANT INFORMATION ABOUT YOUR



THIS SCOOTER SHIPS WITH ONE OR MORE BATTERY TERMINALS DISCONNECTED.



- Unzip both sides of your battery bag, find the disconnected terminal(s) and connect the loose wire harness connector(s).
- Make sure all 4 terminals are securely connected.
- Zip the battery bag back up and connect it to the scooter.
- Now your Travel Pro is now ready to go!
- If once connected your scooter is beeping 5 times after the key is turned on flip the yellow lever in the rear of the scooter.